

UShape II

Golden standard for non-invasive
body shaping Focused

Focused pulsed ultrasound for selective
fat cell destruction without downtime.



UShape II



Golden standard for non-invasive body shaping

U-Shape II uses high-intensity focused ultrasound to permanently destroy fat tissue without harming the surface of the skin.

The U-Shape treatment is clinical proven for noticeable results all over the world.

During a treatment, high-intensity ultrasound energy is focused on the targeted fat, similar to the way a magnifying glass works. The energy passes through the skin and the targeted fat beneath the skin is permanently destroyed.

Over the next few weeks, the body naturally removes the destroyed fat from the treatment area, which leave the patient with a thinner, more contoured waistline.

The U-Shape II Mechanism of Action

Anatomical Depths


Epidermis~0.04mm

Dermis range
~2.5-3.0mm

Subcutaneous
adipose tissue

Normalized
power

● 100%

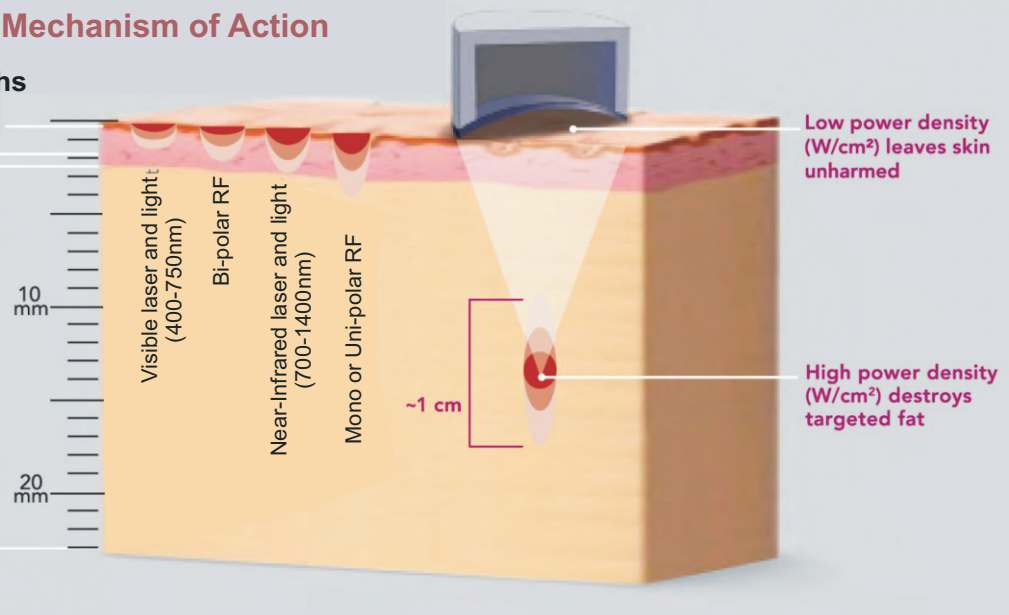

- Fat Reduction
- Skin Tightening
- Body Contouring
- Mechanical Massage
- Metabolism Acceleration

HOW IS U-Shape II DIFFERENT?


✓ Non-Surgical - Non-Invasive

Invasive fat reduction procedures, such as liposuction, typically carry certain surgical risks and are associated with significant post-treatment recovery downtime. Even newer, minimally-invasive techniques are still surgical procedures. Unlike these types of surgical or lasers fat reduction procedures, the U-Shape II treatment is non-invasive and patients typically resume normal activities immediately after the procedure.


✓ Custom Contouring

Patients come in all shapes and sizes, and so does their fat. Other non-invasive body contouring devices can be limited in their ability to treat certain areas of fat and may require several hours and/or multiple treatment sessions to adequately address problem areas. The U-Shape II system is highly versatile, allowing control over precisely how and where energy is delivered, for a truly customized treatment in a single one-hour session.

Control

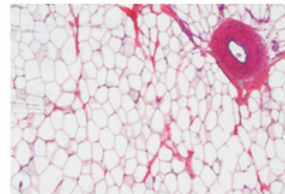


Immediate selective fat cell destruction

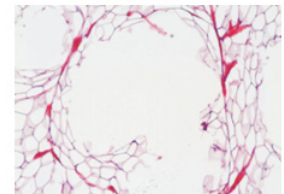

Pre-clinical: Lahav C.R.O.

✓ Real Results - Targeted Efficacy

U-Shape II was designed with body contouring in mind - it's a targeted technology that's tailor-made to deliver amazing results in a single treatment to as large or small an area that you want to address.

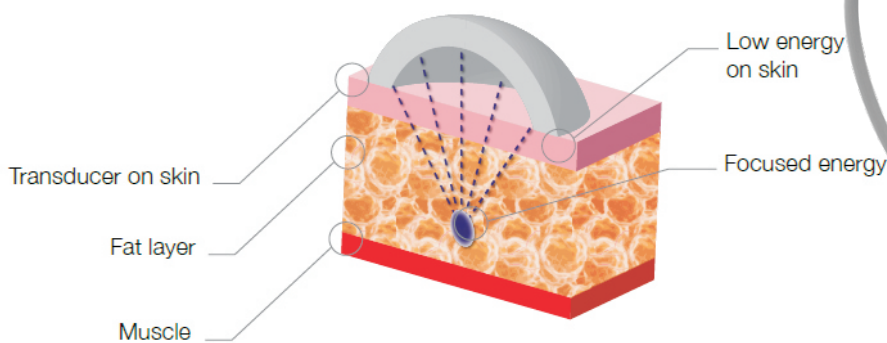

UShape™ II

Advantage

- Quick
- No pain
- Non-invasive
- No downtime

What you can expect from U-Shape II?

Non-invasive / Quick / No pain / No downtime

Immediate results could be seen from every treatment and will also continue to lose weight within 8 to 12 weeks (which is the time needed for the body to naturally process the treated fat).

U-Shape II Procedure



- ★ The treatment head of the U-shape II system is placed on the area to be treated.
- ★ High-intensity ultrasound energy focuses on the targeted fat, destroying fat cells beneath the surface layers of the skin.
- ★ A continuous lesion is created in the targeted fat and the skin layer is not harmed.
- ★ And where it goes? The treated fat tissue is removed through the body's natural healing and elimination process, leaving you with a thinner, more contoured waistline.



Genuine leather

Fat cell

Muscle

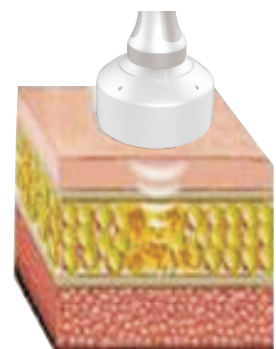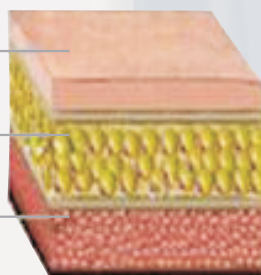

Why choose PZ LASER U-Shape II?



Patented Lock Frequency Technology

Frequency consist at 250KHZ, highest energy at China Market(30% higher than similar products)

PZ Company Patent Technology

Piezoelectric ceramics hemispherical transducer (Material imported from Korea) ;The life span could up to 500,000 shots. American super-silent pump and the ABS sealed double-layer tank.

Intelligent Record Management

The handle has intelligent chip inside. It can record used shots and remains, and the operators can trace record usage by inquiry system, which offers a good reference for stock in advance, without influencing treatment requirements.



Advanced Graphical User Interface

Concise and beautiful Ability to quickly grasp operation treatment.



Adjustable Energy Output

U-shape II has 18 energy levels, which allows physicians to adjust the energy output based on the thickness of targeted fat to be treated and the sensation of the patients. It delivers a safe, effective and comfortable option for fat reduction with high patient satisfaction.

Unique Automatic Computing System

The shots setting is operated by an unique automatic computing system. Setting the length and width of the localized treatment area, the system calculates the total shots automatically adherence to the treatment area. The automatic computing system ensures complete energy delivery over the entire treated area.

Who U -Shape II treat?



The best U-Shape II patients are typically those who have worked to keep their ideal shape, but need some help getting rid of stubborn fat.

Candidates for U-shape II treatment must:

- ✓ Be able to "pinch-an-inch" (2.5 cm) of fat in the treatment area
- ✓ Have a body mass index (BMI) equal to or less than 30
- ✓ Have good overall health
- ✓ Have realistic expectations about the result

Our weight loss technology has been in many clinical research validation and peer-reviewed articles. Tens of thousands of satisfied customers have been and still benefit from the performance and the standard patient satisfaction.


Before and After Results




Before



After

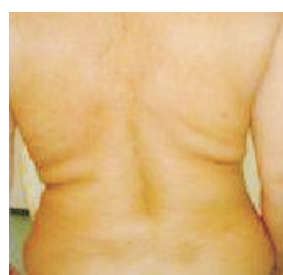

Before

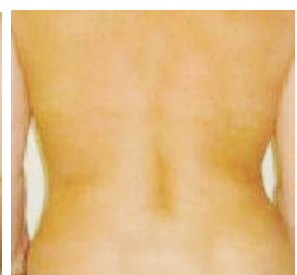

After



Before



After




Before



After


SPECIFICATION

U-Shape Parameters	
Input power:	500W
Output power:	150W
Energy level:	5-18 continuously adjustable
Pulse Interval:	0.5-3S adjustable
Frequency:	250KHZ
Cooling system:	Closed cycle water to air heat exchanger
Power supply:	AC220V 50Hz/AC110V 60Hz
Package Dimensions:	125*55*58cm
Net Weight:	20kg
Working Temperature:	+15°C~+30°C
Storage temperature:	-20°C~+55°C
Relative humidity :	≤80%
Atmospheric pressure:	86kPa~106kPa

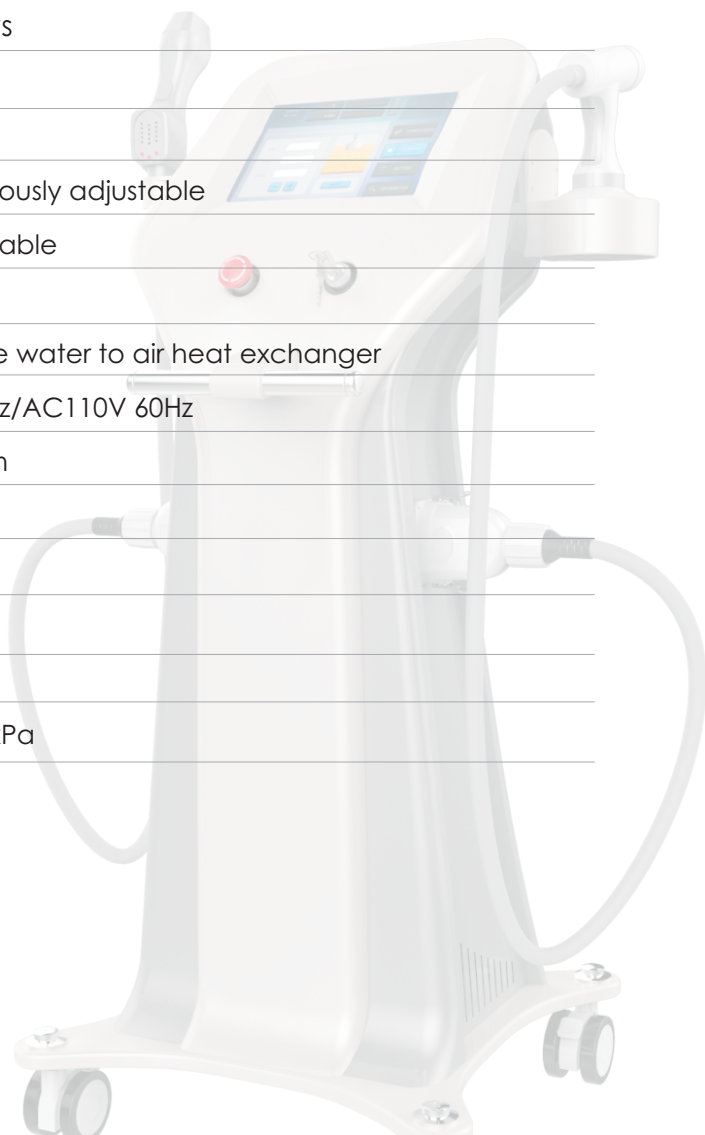

About PZ Laser Company

PZ Laser a worldwide aesthetic device manufacturer produces high-end products for plastic surgeons, dermatologist, physicians and healthcare professionals. PZ Laser serves the industry as one of the leading companies with rigorous R&D level and valuable experience with 15 years histories. PZ Laser products are being globally marketed by overseas offices and worldwide distributors partners.

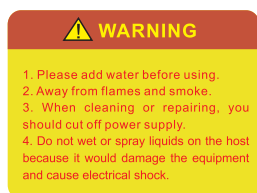

Tel: 0086 - 371 - 55677868
0086 - 371 - 55677886
0086 - 371 - 55011989

Fax: 0086 - 371 - 55677886
E-mail: info@pzlaser.com

📍 Headquarters:

Room 20, FL 4th, Unit 2, Building 13, Electronic Industrial Park, Lianhua Road, Hi-tech Development Zone, Zhengzhou, Henan, China

For more information: www.pzlaser.com